

DOOR BEACON

Model DB10L

Installation of 9-volt Alkaline Battery:

Your "Door Beacon" is powered by a 9 volt Battery (purchased separately). Use only Alkaline Battery. The battery compartment is located in the bottom of the unit. To remove the cover, push in the direction of the arrow. Connect 9-volt battery to the terminals. Position battery into the recessed compartment and replace the cover.

To mount unit on the door:

Unit should be positioned on the inside of the door of the room you will be in.


An "Over the Door" Hanging Bracket is included that allows you to easily hang your "Door Beacon" on any standard door. It also allows the convenience of moving the "Door Beacon" to a different door.



Insert Hanging Bracket from bottom as shown.

Your "Door Beacon" can also be hung in place on the door by installing a screw where you want the unit positioned. (Position high onto door at eye level) Hang the "Door Beacon" onto the screw by using the hole on the molded bracket at the back of the unit.

To attach the "Door Beacon" to the door with the supplied Velcro, separate the two Velcro pieces by pulling apart. Attach the thick, plush piece of the Velcro to the inside of the door after removing the paper backing. Press the adhesive side firmly against the door. Secure the other section of Velcro to the back of the "Door Beacon" unit, first removing the paper backing and then pressing the adhesive to the unit. The "Door Beacon" can then be secured by pressing the Velcro on the unit against the Velcro on the door.

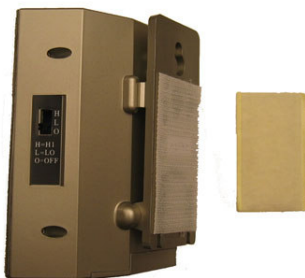

Attach Velcro on back as shown

To use the Hi/Lo/Off Sensitivity Switch:

The switch on the side of the unit marked H, L, and O allows for user adjustment. In the H (High) position, the sensitivity to the vibration of a knock is greater. This would be used on a thick, heavy door as an example. A thinner, lighter door might best have the switch in the L (Low) position because of lower sensitivity. Adjust the switch to suit your personal requirements to achieve maximum results.

It is suggested that the unit be turned to the O (OFF) position when not in use. When unit is in the OFF position, the light will not flash.

Before sending for Service:

If your "Door Beacon" ceases to operate, check & replace the battery if required.

ONE YEAR LIMITED WARRANTY

Your "DOOR BEACON" is warranted to be free from defects in material or workmanship for one year from the date of purchase.

This warranty does not apply to any unit that has been subject to misuse, negligence of accident. At its option, Global Assistive Devices, Inc. may select to replace the entire unit rather than repair it. For Service, return the device with dated proof of purchase, in a secure package, insured and shipping prepaid. Please include a note describing exactly what is wrong with the unit. Include \$9.95 (check, money order, Visa, Master Card, Discover, or American Express) for return Shipping within the U.S.A. (other countries contact us for a quote). Send unit to:

GLOBAL

ASSISTIVE DEVICES, INC.

**1121 East Commercial Boulevard
Fort Lauderdale FL 33334**

Disclaimer: In no event shall Global Assistive Devices, Inc. be held liable for any special, incidental, indirect or consequential damages whatsoever including, but not limited to, damages for loss of property, personal injury, loss of privacy, failure to meet any duty, negligence, and any other loss whatsoever, however caused, even if Global has been advised of the possibility of such damages. This disclaimer of liability applies to any damages or injury, including but not limited to those caused by any failure of performance of the product. Responsibility for selection, safety and suitability of any product rests with the consumer. Circumstances or consequences arising out of or in any way related to the use of or inability to use our products is the responsibility of the consumer. Global Assistive Devices, Inc. cannot be held liable in any way.