

# DOOR BEACON DOOR KNOCK TRANSMITTE

## MODEL DBT300

Open Battery Compartment Cover and connect one 9 volt Alkaline Battery to terminals. Insert Battery into compartment and reposition cover.

To attach the included "Over the Door" Wire Hanging bracket, Insert from bottom of unit and slide upward until bracket is secure in slots at bottom of unit. Unit may also be positioned on a door by using the Velcro included. Unit should be positioned on the inside of the door.

Slide switch to ON position to activate. Select High or Low Sensitivity. Unit should be in Off position when not in use.

Easy to set Selectable Frequencies are located inside battery compartment. Set the numbered dip switches to match the receiver dip switches being used. Different settings can be used to allow for use of more than one transmitter/receiver system without interference.



To replace the light bulb: (290 ma, 4.8 volt required; one replacement included)

- 1) Remove 4 screws in back of cabinet.
- 2) Remove Battery Compartment Cover and Cabinet back.
- 3) Remove screw in middle of circuit board
- 4) Lift printed circuit board.
- 5) Unscrew bulb (small pliers may be required)
- 6) Install replacement 290ma, 4.8Volt Light Bulb.

## ONE YEAR LIMITED WARRANTY:

**DOOR BEACON DOOR KNOCK TRANSMITTER Model DBT300** is warranted to be free from defects in material or workmanship for one year from the date of purchase. This warranty does not include light bulb. This warranty does not apply to any unit which has been used for other than its intended use, or has been subjected to misuse, negligence or accident, or has been opened or serviced by anyone other than an authorized GLOBAL ASSISTIVE DEVICES, INC. Service Center. GLOBAL ASSISTIVE DEVICES, INC. may select to replace the entire unit rather than repair it. Before sending unit for service, check to ensure that battery and/or light bulb do not need replacing. For service during the warranty period, return the unit, with proof of purchase date, in a secure package, insured and with shipping charges prepaid. No C.O.D. packages will be accepted. Include your return street address and a note describing the service required. Enclose \$7.50 (check, money order, Visa, Master Card or American Express) for domestic shipping and handling charges. Other countries please contact us for pricing.

GLOBAL ASSISTIVE DEVICES, INC  
Fort Lauderdale FL USA