

PUSH BUTTON TRANSMITTER Model PBT300

Installing Battery

Your door transmitter button operates with an Alkaline 23A 12V or equivalent battery. To install battery, insert a coin in slot at bottom of the casing. Turn coin to release the front from the back of unit. Insert battery as marked (matching + to positive and – to negative).

To fasten the unit near the door

With front cover off, position back of unit in a suitable location near or on the door. Fasten in place by inserting and securing screws through the two holes provided. Replace the front and snap into closed position.

Selectable Dip Switches for Code Settings

Your unit comes with the dip switches preset to match the receiver code setting on the DOOR BEACON RECEIVER, Model DBR300. These settings are adjustable and can be changed to avoid interference. The transmitter code must match the receiver code for reception of transmission to occur.

**PushButton
TRANSMITTER**



Operating the Transmitter

To send a signal to a remote receiver to DOOR BEACON RECEIVER, Model DBR300), press the button on the front of the transmitter. The red light indicator will illuminate when the transmitter is sending a signal.

FCC Compliance

This transmitter complies with the FCC rules and Regulations and because of this has limited transmission power and range. Any modifications or changes to this device will void the warranty and may void compliance with the FCC Rules and Regulations. Operation of this device may not cause harmful interference and must accept any interference received, including interference that may cause undesired operation. Radio signals on or near the receiver operating frequency may block the transmitter signal regardless of the code setting. If this occurs the radio receiver may have to be repositioned. The receiving device cannot respond to more than one transmitted signal at a time.

ONE YEAR LIMITED WARRANTY:

WIRELESS PUSH BUTTON TRANSMITTER, Model PBT 300 is warranted to be free from defects in material or workmanship for one year from the date of purchase. This warranty does not cover battery replacement or water damage. The warranty does not cover any unit which has been used for other than its intended use, or has been subjected to misuse, negligence or accident, or has been serviced by anyone other than an authorized GLOBAL ASSISTIVE DEVICES, INC. Service Center. GLOBAL ASSISTIVE DEVICES, INC. may select to replace the entire unit rather than repair it. Before sending the unit for service, check to ensure that the battery does not need replacing. For service during the warranty period, return the unit, with proof of purchase date, in a secure package, insured and with shipping charges prepaid. No C.O.D. packages will be accepted. Include your return street address and a note describing the service required. Enclose \$9.95 (check, money order, Visa, Master Card, Discover or American Express) for domestic shipping and handling charges. Other countries please contact us for pricing.

GLOBAL ASSISTIVE DEVICES, INC.

1121 East Commercial Boulevard, Oakland Park FL USA 33334