



## MODEL SL12

The PRISM Strobe Light is designed for use with alerting devices with a 12 Volt DC output jack. A power source cable is included to connect the PRISM to the alarm clock or alerting device. When connected, the PRISM will flash when the alarm clock or device is activated.



Insert the barrel plug into the PRISM and plug the 3.5mm plug into the clock or device.

Place the PRISM for optimum viewing. It may be placed on the night table near the alarm clock or mounted using the attached bracket and a screw.

Open the compartment at the bottom of the unit for access to the Dry Contact input jack. This is designed for a Hardwired application of other alerting devices.

### **Before sending for Service:**

If your PRISM does not operate, check that the PRISM is receiving voltage from the alarm clock or device that it is connected to.

### **ONE YEAR LIMITED WARRANTY:**

Your PRISM is warranted to be free from defects in material or workmanship for one year from the date of purchase.

This warranty does not apply to the strobe tube or to any unit that has been subject to misuse, negligence or accident. At its option, Global Assistive Devices, Inc. may replace the entire unit rather than repair it.

For Service, return the device with dated proof of purchase, in a secure package, insured and shipping charges prepaid. Please include a note describing exactly what is wrong with the unit. Include \$8.50 (check, money order, Visa, Master Card, Discover or American Express) for return Shipping within the U.S.A. Send unit to:



**GLOBAL ASSISTIVE DEVICES, INC.**  
4950 North Dixie Highway  
Fort Lauderdale FL USA 33334-3947